

Central Dental is pleased to announce that we are now an authorized distributor for Dental Wings in Canada.

Dental Wings is a Canadian company based in Montreal established in 2007.

Dental Wings is a world leader in Digital Dental Technology. Two Canadian companies now working together to bring you the latest in digital innovative products along with strong ongoing support and service.

7Series



Price Lease from **\$630.00**
per month
\$29,500

3Series



Price Lease from **\$405.00**
per month
\$19,000

dental wings
scan&design systems

Available software applications

SPECIAL OFFER
Receive 2 extra years of free
support & upgrade
after first year of warranty!

\$5,000 value
7Series

\$3,000 value
3Series



guided surgery



implants



crown & bridge



model builder



full dentures



partial
frameworks



bite splints



orthodontic
archiving

For further information please contact Ljepa or Iain @ 1.800.268.4442



SUCCESS INJECTION SYSTEM & LUCITONE FRS STARTER KIT

PROVEN ACCURACY: Success Injection System gives you the best fitting denture, precise rearticulation and saves you time with occlusion adjustment compared to traditional packing.

Versatility In One System

**SPECIAL PRICE
\$8,499**

- **Heat-Cured**
- **Microwave**
- **Pour Resin**
- **Silicone Soft Liner**
- **Thermoplastic**
- **Flexible Partial**

**PURCHASE a Success Injection System
#903090 (Aluminum Alloy or Microwave Flasks)
and Associated System/Flask Components
(Special Price of \$8,499),
RECEIVE A LUCITONE FRS STARTER KIT FREE!
(VALUED AT \$2,887)**

Cannot be combined with any other DENTSPLY offer.



SUCCESS INJECTION SYSTEM

Success System consists of the Injection Unit, aluminum alloy or microwave Flasks, and associated system/flask components.



LUCITONE FRS STARTER KIT

With Lucitone® FRS® you can offer a range of SPECIAL PRICE, flexibility, from a firmer resilient design all the way to a super flexible partial.

VIRTUAL® XD EXTRA DEFINITION

Make your first impression
your best impression!

Introducing Virtual XD, an advanced vinyl polysiloxane impression system featuring "Xtra Definition" wash materials providing:

- Optimized flow characteristics for deeper penetration into the sulcus and improved coverage of the preparation.
- Advanced wetting ability for precise detail reproduction of both soft and hard tissue.
- High tear strength to maintain the integrity of preparation margins and fine detail upon removal of the impression.

Try Virtual XD today to see your next case in Xtra Definition!

BUY: 3 REFILLS

GET: 1 REFILL FREE

BUY: 6 VIRTUAL XD 380 REFILLS AND 8 VIRTUAL XD WASH MATERIAL REFILLS

GET: 1 VIRTUAL MIXER FREE

Virtual® XD VPS Impression Materials

Material	Product	Set Time	Product Code
Tray Materials	Virtual XD – 380ml	Regular Set (4:30)	646449
		Fast Set (2:30)	646448
Wash Materials	Virtual XD Extra Light Body – 50ml	Regular Set (3:30)	646468
		Fast Set (2:30)	646467
	Virtual XD Light Body – 50ml	Regular Set (3:30)	646462
		Fast Set (2:30)	646461

Virtual® VPS Impression Accessories

Accessory	Product	Size/Color	Product Code
Mixer	Virtual Mixer	120V, 60Hz	655988
Mount	Virtual Mixer Wall Mount	Compact Wall Mount	655987
Mixing Tip	Virtual 380 Mixing Tip – 50 ea	Yellow Base	600158
Bayonet Ring	Virtual 380 Bayonet Ring – 2 ea	Yellow	600159
Tray Adhesive	Virtual Tray Adhesive	10 ml	562841AN



Model MT10

The MT10 offers all the features of the MT12 in a compact design. It is the ideal fit in the close quarters of a doctor's office laboratory, and also performs very well in the demanding environment of a commercial laboratory.

Specifications

- 1/3 HP motor, with 1.35 service factor producing .45 HP actual power
- 10" diameter, 1/4" thick grinding wheel
- 4" wide, 4-1/4" high cutting surface
- 14" high, 14" wide, 15" deep
- 115 volt, 60 Hz standard or 230 volt, 50/60 Hz optional for export
- Net weight 32 lbs, shipping weight 38 lbs.



Ray Foster

Price

\$695.00

Item: MT10



Price
\$6.95

Item: A901CSA

Sperian Fuse Safety Glasses

Sperian Fuse has a sleek gunmetal-colored metal frame with exceptional coverage. Fuse safety glasses aggressive wrap-around lens design includes adjustable break-away FlexiCord. Dual 9-base wrap-around protection with a metal frame.



Item: 33025

Silicone Mixing Bowl

25 ml mixing bowl with a non stick surface and a suction cup bottom which allows the bowl to stay in position. Suitable for acrylics.

Special
\$11.60
Reg. \$15.95

Pliers

CDP-3 Pliers 3 Prong

CDP-2 Pliers 2 Prong

Price

\$19.95



dentalex MICROSCOPE™ the MICROSCOPE for DENTAL TECHNICIANS


The DENTALEX MICROSCOPE is small, lightweight and portable. It has a robust base for stability and because it is not fixed to the workbench, it can be shared around the lab.

The DENTALEX MICROSCOPE requires no additional lighting and no power source. Setup is a snap and operation is easy. Focusing is straightforward and quick.

**Uncomplicated,
Simple to Use**

Special
\$1085.00
Reg. \$1175

**Getting Close
To Your Work...**

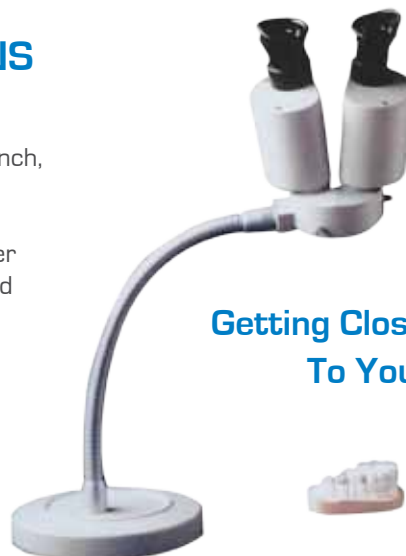



SAVE 40%

AcryLux working mould guide
53 Sets + Shade Guide (Plastic Case)
\$387.40

AcryPlus working mould guide
42 Sets + Shade Guide (Plastic Case)
\$572.00

AcryPlus EVO working mould guide
38 Sets (2 shades) + Shade Guide (Metal Case)
\$923.00

Acry Tray LC

Bases of light cure resin, that are very simple to use, have excellent dimensional stability and twisting resistance.

Acry Tray LC bases are indicated for:

- Individual impression trays
- Functional impression trays for cases of edentulism
- Bases for mounting teeth in removable dentures
- Bite plates

\$59.95

Item: 0800001
ACRY TRAY LC 50 Pcs



Alginate

Alginate for high precision dental impressions. Available in both Chromatic and Dust Free. Packaging of 12 bags of 453g each.



Chromatic:

Alginate with colour changes.

Flavour: Fruit.

Chromatic variations: White – Mixing.

Violet – Working.

Pink – Mouth insertion.



Dust Free:

Fast mixing and setting time.

Flavour: Strawberry.

Colour: Pink.



Special
\$6.95 /each
Reg. \$9.95

**Buy 12
& Receive
FREE
Bowl &
Spatula!**

2040001 Alginate - Chromatic 453 g
2060001 Alginate - Dust Free 453 g

EXTRACTOR with activated charcoal filter



Item: 20192

Price

\$1495.00

Height: 260mm x Width: 220mm
Depth: 260mm | Weight: 6kg (230V/50Hz, 100W)

> Protects against hazardous vapors

Right from the start you will be able to avoid the health risks of hazardous vapors. This user-friendly exhaust system allows you to target the ever-increasing risks from chemical substances.

The degree of separation for insulation fluid is 84%, for Monomer 89%, if the exhaust system is used with the activated charcoal filter, and if the filter is regularly replaced after approx. 50 operating hours.

A light sensor automatically turns on the system during countertop operation. Upon completion of the work process, the system will turn off with some time delay.

5+1

RONCO'S PREMIUM LINES OF EXAMINATION GLOVES

NITECH®

RONCO NITECH®

- Patented blend of premium quality synthetic materials
- Powder free



NITRILE

RONCO NITRILE

- Diamond textured for superior grip
- Powder free



RONCO TOUCH

- Premium quality nitrile in pink colour
- Powder free



LATEX

RONCO SILKTEX

- Superior barrier protection with excellent elasticity
- Available as powder free and lightly powdered



SOAPOPOPULAR Instant Hand Sanitizer

Kills 99.9% of harmful germs!

\$8.49
550 ml
Item: 68014



> **World's #1 alcohol free foaming instant hand sanitizer**

- safer for kids
- hypoallergenic
- gentle to the skin
- non-drying to hands
- no sticky residue
- non-flammable
- fragrance free
- alcohol free

Buy 6
1000ml
Cartridges
& Receive
1 FREE
Dispenser!

\$19.99
1000 ml
Cartridge
Item: 68000

\$19.95
Dispenser
Item: 681111

soapopular



SEMI-ADJUSTABLE
Articulators

\$225
AACPO452
Facebow Elite

\$250
AAR60254
Articulator

ACCURATE TRANSFER



BREIDENT High luster polishing with the handpiece



Polishing of acrylics with the handpiece.
Brushes and buffs for handpieces provide
brilliant high luster on all dental acrylics.



Cotton buff
22 mm
15 Pieces
350 0065 0
Reg. \$61.71
Special \$51.71



Cotton buff
22mm
15 Pieces
350 0091 0
Reg. \$71.43
Special \$61.43



Star brushes goat
hair white
19 mm
15 Pieces
520 0015 1
Reg. \$56.95
Special \$46.95



Leather buff
22mm
15 Pieces
350 0066 0
Reg. \$48.62
Special \$38.62



Star brushes
goat hair white
13 mm
15 Pieces
520 0014 1
Reg. \$52.95
Special \$42.95



Polishing buff
felt 3 layers
22mm
15 Pieces
350 0064 0
Reg. \$40.02
Special \$30.02



Acrypol polishing
paste for veneering
materials 170 g
520 0017 0
Reg. \$31.04
Special \$25.00



Avrasi-Starglanz asg
Universal high luster
polishing paste
2 x 50 ml
520 0016 3
Reg. \$53.86
Special \$45.00

Technical Corner

visio.lign




1 Determining the suitable design for the physiognomy and the residual teeth



6 Try-in and correction if required



2 The position of the anterior veneers is marked

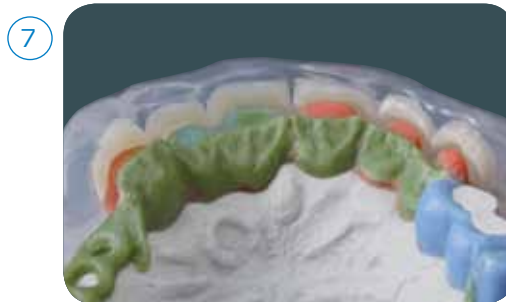

7 Veneers support planning and the transfer of the wax-up of the framework. Phonetic and esthetic aspects are the basis for the optimization of the wax-up.



3 The veneers are fixed using tooth-colored wax, for example beauty setup wax by Bredent



8 Condition with metal primer then opaques




4 Based on bite registration, the anterior and posterior veneers are completed for the esthetic try-in



9 Grind the veneer at an obtuse angle




5 Completed esthetic set-up

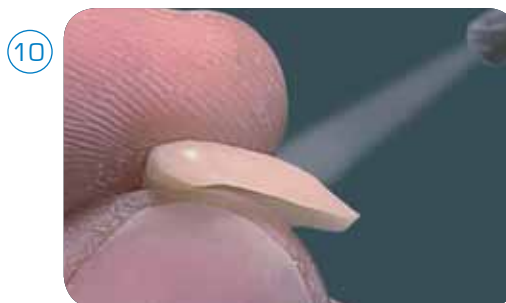

10 Sandblast the inside of the veneers, including the border area

11

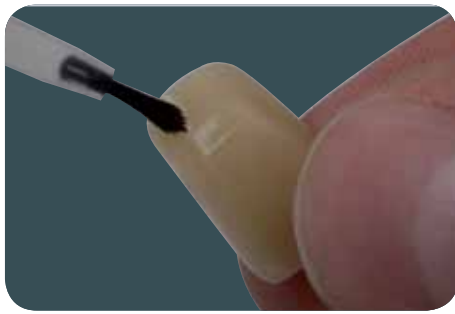

Apply visio.link thinly to the inner side of the veneer & beyond the margins

17



crea.lign is used for contouring in the palatal...

12



Apply combo.lign to the conditioned inner side of the veneer

18



... and the cervical areas, Individual contouring also in the gingival area

13



Excess material obtained by pressing on the matrix does not need to be discarded. It can be used to prepare a bonding layer to the crea.lign finishing material

19




Applied layer of crea.lign with intermediate polymerization. A handlamp can be used for intermediate polymerization or fixation of the layers

14



From the labial side, the veneers are fixed by the transparent visio.sil matrix material. The matrix is removed for final hardening (polymerization)

20

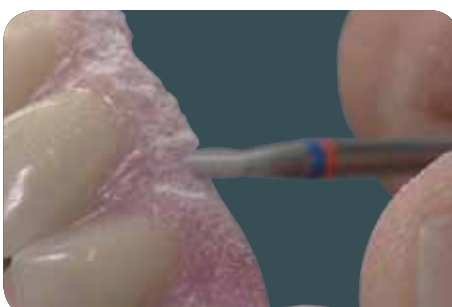

Finishing with a tungsten carbide bur

15



In the cervical and approximal areas, combo.lign which is still paste-like, is removed with an instrument

21



Prepolishing of crea.lign

16



After bonding, crea.lign is applied approximally

22



Prepolishing with a goat-hair brush and Acrypol or pumice



"The **BEST** there is!" Laminated Mouthguards

- ✓ Anterior brace for extra protection!
- ✓ Custom-fitted for great oxygen intake!
- ✓ Easy to attach break-away strap!
- ✓ Custom-made shock-absorbing surface!
- ✓ Professionally made for a perfect fit.
- ✓ Laminated material for superior strength.
- ✓ Excellent abrasion resistance!
- ✓ Made by dentists worldwide.



Proform Machine IV Vacuum Former

Special #7000338
\$475.00
Regular \$507.75

Proform Mouthguard Material

Special
\$68.26
Regular \$75.85
Includes: Solid, 2-Colour & 3-Colour (12/box)



BREDENT Abraso-soft Acryl

ø 80 mm Due to the polishing heat the open pore special fleece and the bleached Chungking bristles absorb more polishing paste and therefore up to 50 % of working time can be saved. Recommended speed: 3,000 rpm 1 piece.



Item: 35000800
Special
\$13.95
Reg. \$17.20

BREDENT Set-up grinding tool

Grinding without exchanging tools in a single working step. (Grinding tip: 80 - 100 µ, wide grinding part: 160 - 180 µ). The small, precisely shaped grinding tip with fine, perfectly cutting diamond grains provides the ideal precondition for well-aimed and rapid grinding in of occlusal contacts.
1 piece.



Item: 34001010
Special
\$79.39
Reg. \$89.39

FOREDOM K.1030 Portable Micro Motor

This all new design has a pocket sized control unit which features a digital speed display at the top so it's easy to read and monitor the speed, which is easy to control with the dial control on the front. It operates at a speed of 0 - 30,000 rpm.



Item: K.1030

Special
\$435.98
Reg. \$458.90

RAZERTIP Handpiece

Especially designed for cutting and trimming all dental materials. Ideal for bleaching trays, mouthguards and all other types of appliances. Variable heat control.



Item: SS-DHD

Price
\$199.95



High Arch Brass Articulator

Price
\$10.95

Item: 105015



Compress #15 & Flasks Combo



Special
\$224.00

Reg. \$274.00

Item: 44AUPPER & 44ALOWER
#15 [COMPRESS]



Reline Jig

Special
\$31.95
Reg. \$39.95

Item: 113145



Mixing Tips

YM100YB - Yellow Mixing Tips (50/Pkg.)
YM110PB - Pink Mixing Tips (50/Pkg.)
YM120GB - Teal Mixing Tips (50/Pkg.)
YM130PB - Purple Mixing Tips (50/Pkg.)

Price
\$20.00



RONCO - Face Masks

Price
\$5.25

Item: #5614



Bibs

All Colours!

Special
\$19.95
Reg. \$23.95



23-1210 Bibs Blue (500/Pkg.)
23-1211 Bibs Green (500/Pkg.)
23-1212 Bibs White (500/Pkg.)
23-1213 Bibs Rose (500/Pkg.)
23-1214 Bibs Yellow (500/Pkg.)
12-1215 Bibs Lavender (500/Pkg.)

Coe Tray Cleaner

Special
\$34.65
Reg. \$38.50

Item: 250010
Bottle (575g)



Border Lock Trays Autoclavable Edentulous

Special
\$225.95
Reg. \$249.95
Item: 186800



UPCOMING COURSES & EVENTS

Denturist Association of Canada

2014 Annual Meeting Aug. 4 - 9th - St. John's Newfoundland

Impression Taking And Soft Tissue Model Course

Hands on course using live patient and verification jig

Sept 12, 2014 - Lethbridge AB

Implant Supported Fixed Bar Prosthesis

Hands-on Lab and Clinical based training

with Live Patient experience

Sept 13 -14, 2014 - Lethbridge AB

2014 Perfecting Your Practice

The Denturist Association of Ontario PYP Conference

Sept 18 - 20, 2014 - Mississauga ON

Spectrum Vancouver

Sept 19, 2014

Vancouver BC

Dental Technology Academy

DTA Lecture Series

Sept 20, 2014

Scarborough ON

Expodent

Sept 25 - 27, 2014

Montebello Quebec

Denturism Implant Study Club

DISC Lecture Series

Sept 27, 2014

Scarborough ON

Dentechnica du Quebec

Oct 8, 2014

Montreal Quebec

Spectrum Toronto

Oct 24, 2014

Toronto ON

Dental Technology Academy

Evaluating Investments Opportunities for Your Lab

DTA Lecture Series

Nov 08, 2014 - Scarborough ON