



	Regular	Special
Linda 3 Chair	\$8,695.00	\$7,825.00
CLX-110 Light	\$1,295.00	\$1,230.00
Bracket for Linda 3 Chair	\$450.00	\$385.00
146-206 Light Post 54"	\$189.00	\$175.00
C-4250 Cuppidor Asepsis Design with Vacuum, Accessory Group and Water QD	\$1,490.00	\$1,380.00
	Package Special \$10,995.00	

*For memory foam an additional \$800

LINDA 3 Chair

SPECIAL DENTURIST PACKAGE!

Lease From
\$235
Per Month / 5 Year Term



A-Silicone VPS Putty

CAD/CAM A-Silicone Laboratory Putty

Scannable A-silicone putty for laboratory applications. Recommended for all CAD/CAM system users for producing scannable bite regulation keys. In combination with Platinum 95, ideal to create high precision masks using the pressure technique and to obtain artificial gums or elements made from high quality composite material

- Total setting time 4:00 minutes
- Shore-A 75 final hardness

**BUY 2 packs of Platinum 75
CAD GET 1 of the same FREE!**



\$169³⁰

C400741 **Platinum 75 CAD** – 1 lb - Standard Pack (1x 800 g tub base / 1x 800 g tub catalyst / 2x spoons)

VPS Duplicating material

Elite Double

A complete range of Zhermack A-Silicones for all types of duplications allowing every need of the dental technician to be satisfied. The product range consists of nine products with four different final hardnesses and three setting times: normal, fast, and the innovative extra fast, designed for automatic mixers. Formulated high degree of opacity, have been optimised to facilitate the reading of even the thinnest thicknesses

- Total setting times: - Regular Set 20 minutes - Fast Set 10 minutes
- Available in 8, 16, 22, 32 Shore Hardnesses



Standard Pack (1x 1 kg tub base / 1x 1 kg tub catalyst (4.4 lbs total))
Economy Pack (1x 5 kg tub base / 1x 5 kg tub catalyst (22 lbs total))

\$149⁸⁰
\$695⁰⁰

**BUY 2 Elite Double
(Any Size) GET 1
of the same FREE!**



Doublemix

DOUBLEMIX is the new automatic A-Silicone mixer for duplications developed by ZHERMACK offering speed and constant quality in all daily duplications.

Its easy handling and ease of use with elite double extra fast (1kg, 5kg, 10kg) make DOUBLEMIX extremely versatile. Its simple, modern design integrates perfectly into all types of dental laboratories and the solidity of its base materials are guarantee to last longer over time.



C309021

\$2600⁰⁰

**Purchase Doublemix Duplicate Machine and Receive 2
Elite Double A-Silicone (1 kg. Packages) FREE!**

\$300 Savings!

1x Doublemix / 1x Quick Connection Set to 1 kg bottles / 48x Yellow mixing tips / 2x Bayonet filters / 1x Working area support plate / 1x Allen wrench

Gingival mask material

Gingifast

Elastic type

Laboratory silicone for gingival reproduction for models in fixed prosthesis.

Rigid type

Laboratory silicone for gingival reproduction of implant prosthetic models.

- Total setting time 10:00 minutes
- Shore-A final hardness - Elastic - 40 - Rigid 70 (after 1 hour)



**BUY 2 packs of Gingifast
GET 1 FREE!**

\$71⁵⁰



- C401500 **Gingifast Elastic – soft type for crown and bridge**
2 x 50ml cartridges, 12 mixing tips, 12 oral tips, Separator 10ml – 40 Shore-A
- C401520 **Gingifast Rigid – rigid type for implants**
2 x 50ml cartridges, 12 mixing tips, 12 oral tips, Separator 10ml – 70 Shore-A

Gingifast CAD

Scannable A-silicone fluid for the laboratory reproduction of artificial gums. Two hardness versions ideal for silicone gums on models with prostheses, bridges or crowns

- Total setting time 10:00 minutes
- Shore-A final hardness - Elastic - 40 - Rigid 70 (after 1 hour)




**BUY 2 packs of Gingifast or
Gingifast CAD GET 1 FREE!**

\$76⁹⁵



- C203227 **Gingifast - Elastic CAD - Standard Pack**
2x 50 ml cartridges / 12x Blue Mixing tips / 1x 10 ml separator bottle with pump spray
- C203232 **Gingifast - Rigid CAD - Standard Pack**
2x 50 ml cartridges / 12x Blue Mixing tips / 1x 10 ml separator bottle with pump spray

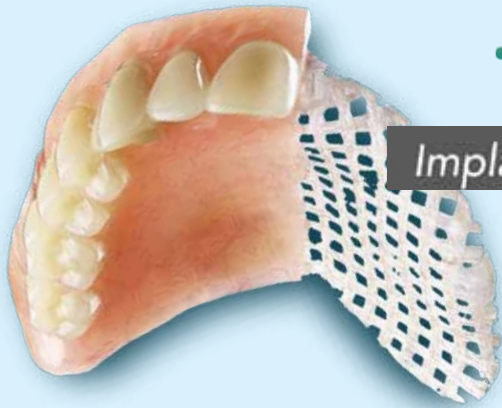

Implant Overdentures **NEED TO BE REINFORCED!**

Removable implant retained overdentures on two to four attachments provide a much improved removable solution for patients. Yet due to the increased masticatory forces applied these overdentures are prone to breakage, especially at the attachment sites. The traditional method to strengthen them is by incorporating a cast metal framework. While this approach works well it adds bulk and weight to the denture and in many cases is not even possible due to limited available vertical space.

FiBER FORCE® starter kit includes:

- 2 - packs of FiBER FORCE mesh (pink)
- 1 - pack of FiBER FORCE 2mm braided rope (pink)
- 1 - syringe of CST Flow pink light cure resin (3ml)
- 100 - sheets of pink spacer wax (0.5mm)
- 100 - patient brochures (with cardboard stand)
- 1 - EZ VAC vacuum forming unit

HOT DEAL

Buy a FiBER FORCE® Starter kit for \$538 and receive an extra pack of FiBER FORCE® mesh **FREE - SAVE \$109**

Using FiBER FORCE® pre-impregnated glass fiber mesh, you can now create a custom-fitted fiber mesh framework that will provide the same level of reinforcement as a casting without adding any thickness or weight to the denture. Because FiBER FORCE® mesh frameworks are only 0.5mm thin a FiBER FORCE® mesh framework can fit into even the most "vertically challenged" denture.



Everything you need in the FiBER FORCE® starter kit

The best way to get started with FiBER FORCE® in a way that will ensure your success and satisfaction is a Starter kit, which is on promotion for a limited time.

Buy any six packs of FiBER FORCE® at regular price (mesh, UD, 2mm rope, 1mm rope) and get a 7th pack **FREE - SAVE UP TO \$109**

HOT DEAL



HOT DEAL

Buy a CST® Denture Complete kit for \$510 and receive a pack of CST 1:6 fibers (450mm) **FREE - SAVE \$125**



"ALL ON CST®"

CST®, a FiBER FORCE® product, is an ideal solution for reinforcing the transitional denture in immediate-load ALL ON type fixed hybrid denture cases. These dentures are prone to breakage, leading to wasted time and frustration for you, your dentists and your patients. With CST® they can be quickly, effectively and economically reinforced, removing a major source of frustration.

CST® hybrid compressible glass fibers are pre-impregnated in a light cured acrylic/resin, making them easy to use. CST® fibers chemically bond to common methacrylate-based dental acrylics and composites, creating a strong and durable appliance.



Courtesy of Dr. Jay Jurek

CST® complete kit includes:

- 2 - 450mm CST® Pink Hybrid Compressible Fiber (1:6)
- 2 - 300mm CST® Pink Hybrid Compressible Fiber (1:4)
- 1 - 3ml syringe of CST® Flow Pink Resin with tips
- 1 - 5ml bottle of CST® Bond Resin
- 4 - CST® Temporary Support Pillars




Reg.\$418
\$258⁰⁰
34001070

*Limit 1 per customer
for Kit purchases.

Reg.\$275
\$211⁰⁰
34000205

Bredent Diagen Turbos

Diagen turbos are very aggressive sintered diamonds that work effectively on zirconium oxides such as Procera™, Inceram™ and all other ceramic materials. Each stone is moulded to provide optimum grinding properties and cuts effectively down to the last layer.

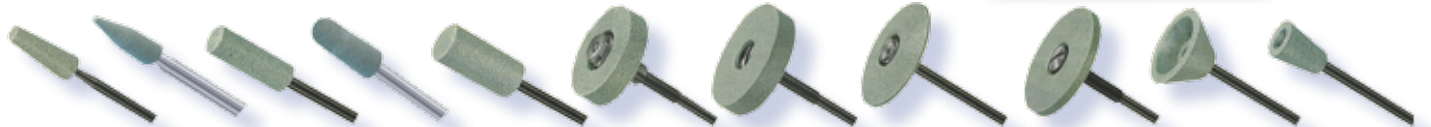

34000150	34000155	34000160	34000165	34000170	34000180	34000190	34000210	34000220	34000240	34000250
\$60.00/2pk	\$60.00/2pk	\$68.00/2pk	\$79.00/2pk	\$85.00/2pk	\$85.00/2pk	\$85.00/ea.	\$64.00/ea.	\$64.00/ea.	\$65.00/ea.	\$63.00/ea.

Turbo Refills Less 15%

Qu-Resin

Quick-setting, self-curing denture repair resin based on diacrylate for intraoral and extraoral (direct and indirect) use.



*New users only
Limit 1 per customer
for Kit purchases.

Special
\$99⁹⁴

PINK KIT
54001161
~~\$149⁹⁴~~

DENTIN KIT
54001162
~~\$149⁹⁴~~

**3 + 1
Cartridge
Refills**

PINK CARTRIDGE
54001165
\$119⁰⁰

DENTIN CARTRIDGE
54001166
\$119⁰⁰

QU-CONNECTOR
54001163
\$29¹⁷

The perfect material when it comes to speed - regardless whether in practices or laboratories. Qu-resin rosa and Qu-resin dentin for standard processing in the laboratory and for direct use in situ. Qu-resin covers a wide indication range thanks to simple handling, quick setting and two colours (pink and dentin).



Ø 80 mm
1 piece

\$14⁹⁵
350 0099 3

Abraso-Sil Acryl

This buff consists of a nonwoven fibre fabric in the centre between two layers of silicone-coated cotton fabric.

On the outside there are two rows of bleached Chungkink bristles. This brush absorbs polishing paste or pumice and releases it slowly for efficient prepolishing.



8 + 1 FREE!

Happy Anniversary

Celebrate with Renfert...



90
YEARS



NEW!



Special
\$795⁰⁰
Reg. \$850

...the 90th company anniversary and the new cost effective Basic eco

- Compact and convenient sandblasting unit
- Precise working procedures and great cost efficiency
- Innovative PerfectView Concept – unique LED technology

Ideas for dental technology...



Waxprofi

The Waxprofi produces improved prosthetic results more efficiently. Instruments and wax are no longer heated separately, resulting in distortion-free wax denture bases and a huge saving in time.



Reg.\$396 **Special**
\$365⁰⁰

Item Code: 1440-1000

+ 3 pcs. Pin Drills



NEW! Top spin

The Top spin provides the advantages of precision, reliability and durability

- ▶ Absolute precision for drilled pinholes with max. rotation accuracy of ≤ 0.01 mm.
- ▶ Suitable for all pin systems thanks to variable drill depth adjustment (0 – 20 mm) and individual drill alignment.
- ▶ Optimal precision control due to individual drill selection (small, medium, large).

Reg.\$1415 **Special**
\$1295⁰⁰

Item Code: 1840-1000



Waxlectric II

Waxing-up is simple, precise and stress-free due to the accurately set, constant and evenly distributed temperature of the instrument tip. Huge saving in time because separate heating of the instrument and material is no longer required.

Reg.\$510 **Special**
\$450⁰⁰

Item Code: 2157-1000

+ 5 – pack Klettfix discs



MT3 w/ Klettfix disc

The cost effective miracle

- ▶ Above average strong powerful with 1,300 W for max. material removal without pressure
- ▶ Easy, fast cleaning due to the tool free removable front door (quick-release fasteners)
- ▶ 100% corrosion free, robust and durable housing

Reg.\$1185 **Special**
\$1099⁰⁰

Item Code: 1808-1000

TECHNICAL CORNER

CREATING A SCAN APPLIANCE WITH LANG'S DUPLICATOR FLASK AND JET XR FOR RADIOPAQUE ACRYLIC



Lang Dental's Denture Duplicator Flask is very useful for making scan appliances.



Start by spraying the internal Flask. Lang's silicone spray works well.



Use a regular set alginate, it gives you a bit more working time.



Mix the alginate to a suitable consistency.



Fit the bottom half of the flask with alginate.



Cover the tooth side of the denture with alginate.



Position the denture into the bottom side of the flask about two-thirds down.



When the alginate is set, trim to remove excess.



Use Lang's Silicone Spray on the alginate to separate the two halves.




Mix the alginate and fill the upper half of the flask.



Suggestion: Use an alginate syringe to fill the fitting side.



Cover the denture with alginate.

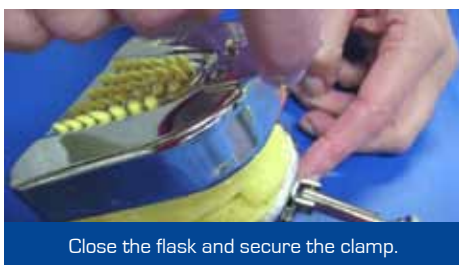

Close the flask and secure the clamp.



Clean up excess.



When the alginate has set, carefully open the flask and remove the denture.



Lang's Dental's JET XR self-curing acrylic with Barium Sulphate; this is opaque with 40%.



Mix the acrylic to manufacturer's specifications.



Use an instrument to dispense the acrylic into the tooth section of the mold.



Give the acrylic a few minutes to "firm up".



Mix clear acrylic and pour into both sides of mold.



Ensure there is adequate acrylic on both sides of flask. Close flask and tighten clamp.

Aquapres®

LANG



Cure the acrylic for 15 - 20 minutes in Lang's Aquapres.



When cured, take out the pot, open and remove.



The cured radiographic guide.



Trim and polish guide.



Mark areas for gutta-percha fiducial markers.



Drill small holes approx. 1-2mm wide, 1mm deep; Heat gutta-percha and push into holes.



... see diagram



You can cover with cold cure if you wish. Alternatively there are radiopaque filling materials that bond to acrylic.



Denture and two different types of guides. The middle one uses clear acrylic for the base and Lang Dental JET XR for the teeth.

The one on the right uses Lang Dental's JET XR 40% for the teeth area (opaque) and 20% for the base (shadow)

Special thanks to Terry Whitty for the step-by-step procedure and images.

THE COMPLETE Implant Duplication System



Ortho-Jet

FEATURES

- Extra fine bead size
- Cross-linking monomer
- Low flow properties
- Superior clarity
- Additional uses



BENEFITS

- Adapts accurately to the mold for a comfortable fit
- Gives the acrylic strength and durability
- Allows better control, will not slump with minimal distortion
- Creates an aesthetically pleasing appliance
- Resists wear—perfect for occlusal splints, night guards and surgical implants



Jet XR™ Self Cure Radiopaque Acrylic

Jet XR™ Radiopaque Acrylic Resin is intended for the fabrication of methacrylate-based acrylic appliances and used wherever a radiopaque acrylic dental device is useful in establishing an anatomical location or site, such as a provisional implant stent.

EDENTA - EXA Technique

**Buy 1 pkg(6)
Receive 1
Polisher FREE!**



(Pkg - 6)

FINE	0651	0652	0653	0654	0655	0656
MEDIUM	0661	0662	0663	0664	0665	0666
COARSE	0671	0672	0673	0674	0675	0676

Keystone - Diamond D

**Purchase 5 lb .
Powder Receive
1 qt. Liquid FREE!**




Reline Jig Spring Type



\$54⁹⁵

02772

STODDARD Brushes

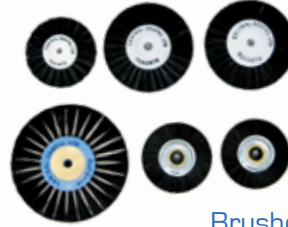

**Purchase 1
pkg(12) Receive
1 brush FREE!**

Brushes: B80/B12/P12/P13

SCHUTZ Flexor Polishers



Special Price

\$39⁹⁵ Reg.\$44.70

658125

(Pkg - 25)

Keystone - Cotton Buffs



**Purchase 1
pkg(12) Receive
2 buffs FREE!**

(4 x 42) 1180170 or (4 x 30) 1180160

Twister Articulator + Intro Kit

- Fully Glue-Less
- 100% Stability
- Steel Like Structure
- Bite Correction With a Twist
- Pindex Model Work
- Digital Printing Models
- Accurate, Practical and Economical

\$85⁹⁵



Includes: 2 twister plus articulators, 2 medium arch rubber bases, 2 large arch rubber bases, 12 patient labels and 24 ball adapters.

Super Ring

For all e.max users:

Now you can press without the worries of a cracked ring!



- Single Ingot or Large Ingot
- Multiple Units or Bridges
- Little to Non-Reaction Layer
- Save time, Investment material and Money

\$59⁹⁵

Lucitone 199 (5 lb)

Lucitone 199 is the premium choice for both tissue and implant-supported full and partial dentures, offering outstanding esthetics, high impact resistance and superb flexural strength.

Special Price

\$279⁹⁹

688120

**2 Liquids
FREE!**

430ml 684309



DENTSPLY

Polyflex & Perflex

**3 + 1
FREE!**



\$82⁸⁸

N021855

7.5 L

Genious Interior Mould Guide



Special Price

\$999⁹⁹

Reg.\$1462

Turbomax Alginate Auto Mixer


**Receive 9 cans
of Jeltrate
Chroma or
Jeltrate Plus!**

AM001

\$1556³³



- Eliminates hand mixing – SAVE time and money
- Exact water/powder ratio every time
- Very efficient – mixes silky smooth in 8-12 seconds
- 6 month warranty

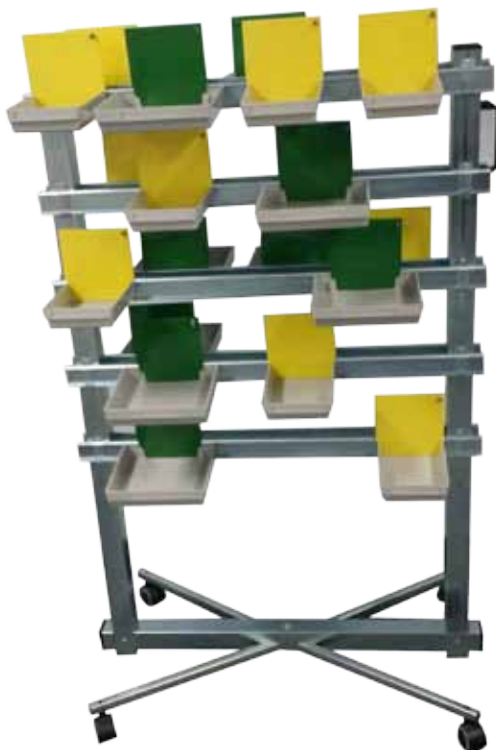

Mestra Lab Pan Organizational Cart

- Organize all your lab parts and accessories
- Save space and time
- Very mobile – moves around easily
- Smart investment

\$435⁰⁰

100048

COURSES AND EVENTS



Digital Implant and Implant
Bar Design (2 day course)

June 12th - 13th, 2015

Location:
Central Dental Ltd.

COURSE OVERVIEW

- Hand's on Design Training
- Single Unit Custom Abutment
- Single Unit Screw Retained Crown
- Bar Design for Single Unit Restorations
- Full Contour Screw Retained Bar Designs

For more info call: 1-800-363-3989

Denturism Implant Study Club DISC COURSES

May 23, 2015

DISC Course

Location:
Central Dental Ltd.

Sept. 26, 2015

DISC Course

Location:
Central Dental Ltd.

Nov. 21, 2015

DISC Course

Location:
Central Dental Ltd.

www.disccanada.ca

Dental Technology Academy DTA COURSES

May 9, 2015

DTA Course

Removable Prosthetics

Location:
Central Dental Ltd.

Sept. 12, 2015

DTA Course

Ceramics

Location:
Central Dental Ltd.

Nov. 7, 2015

DTA Course

Fixed Implants

Location:
Central Dental Ltd.

www.dtacanada.com

Sept. 23 & 26

Denturist Association of Alberta

Convention & Education Symposium

Banff Park Lodge Resort Hotel & Conference
Centre, in scenic Banff, Alberta.

 **Central Dental**
The laboratory and denturist supply specialist.