


cubeX²

Translucent Cubic Zirconia

BEAUTY WITH STRENGTH



Flexural Strength (MPa)

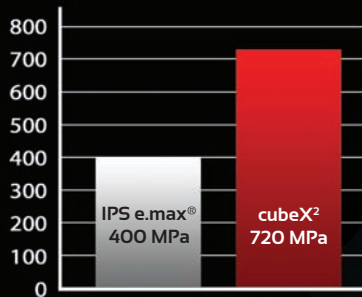

Flexural Strength Testing per ISO 6872 specification.

IPS e.max is a registered trademark of Ivoclar Vivadent.

203-4401 Albert Street, Regina
Saskatchewan, S4S 6B6



Tel: (306) 559-8111
truimage.dl@gmail.com

cubeX²



Translucent Cubic Zirconia

Clinical Indications:

cubeX² is indicated for single crowns and 3-unit bridgework in all tooth positions.

Preparation Requirements:

cubeX² requires less tooth preparation than is required for lithium disilicate. Ideally, a .8 mm chamfer or rounded shoulder prep is preferred; however, a feather-edge margin is acceptable with 1.0 - 1.5 mm occlusal reduction.

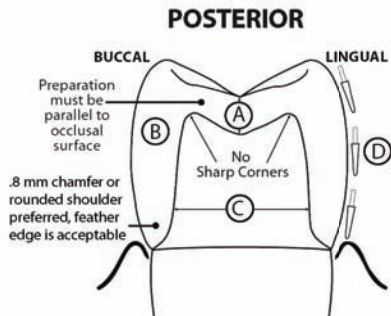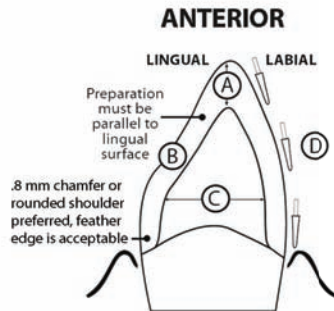

- A. 1.0 mm - 1.5 mm occlusal reduction
- B. 1.0 mm middle third reduction
- C. Buccal and lingual walls must be convergent
- D. Preparation should be cut in three planes